

Attachment B: Monthly Maintenance Checklist for Cardiac Science Automated External Defibrillators

Unit Serial #	Office	POC
---------------	--------	-----

Initial boxes as items are checked off. Keep completed the form on file.

	Jan	Feb	Mar	Apr	May	June	July	Aug	Sept	Oct	Nov	Dec
Step 1 Open Lid												
Step 2 Status indicator should turn red												
Step 3 Status indicator should turn back to green within 5 seconds												
Step 4 Check expiration dates on pads												
Step 5 Listen for voice prompts												
Step 6 Close lid and confirm green status indicator												
Date												
Initials												

Daily Maintenance: verify that the Status Indicator is green.